Ropox Wave Toiletstøtte 135, 40-40125

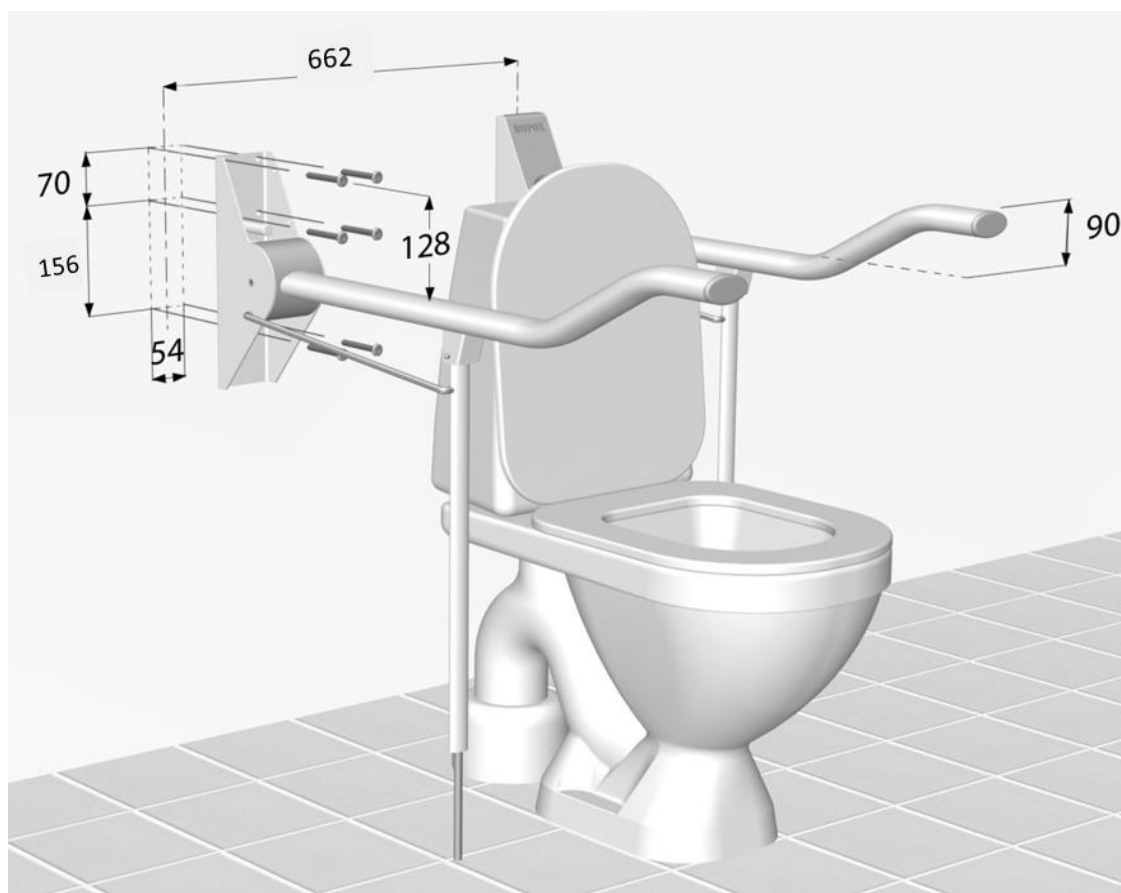
/ Ropox Wave Toilet Support Arms 135

/ Ropox Wave Toilettenstütze 135

Montageanvisning / Mounting instruction / Montageanleitung

I henhold til / according to DS 3028 (Danish Standard)

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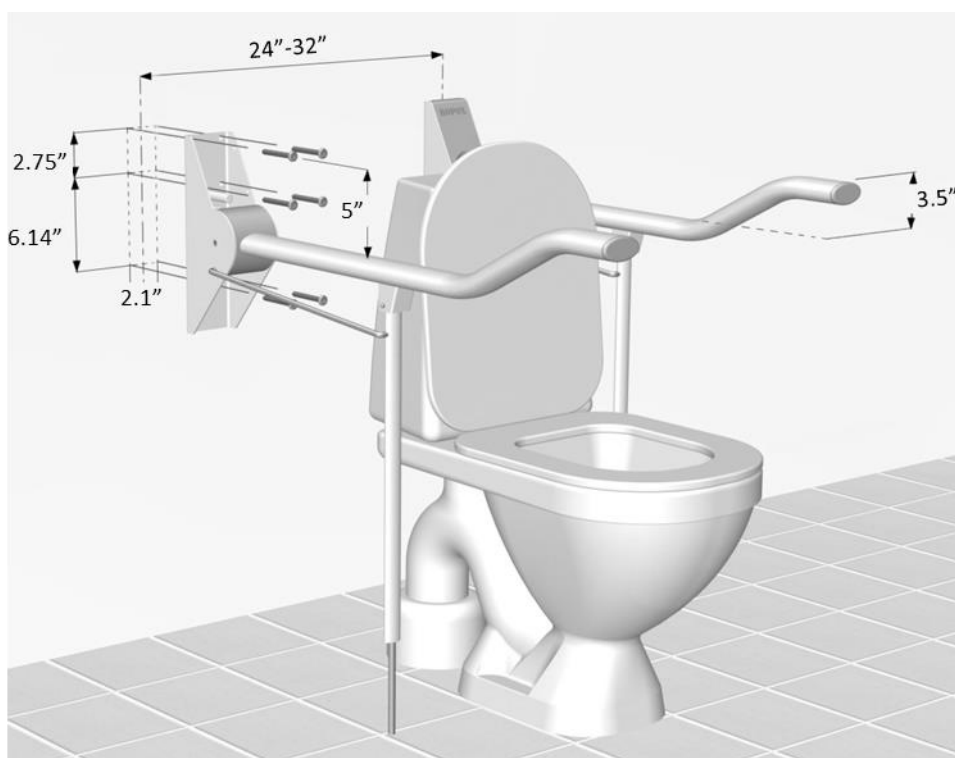
Please check with the user for individual demands which could affect the mounting position. Also make sure that national and/or individual state statutes are fulfilled

Please find attached mounting template showing position of the 6 holes for easy installation

Recommended user heights and installation heights (upper holes in mounting template/bracket)


User height lowest part of wave (recommendation)	Installation height (upper holes)	Situation
30in	33in	Ropox Wave Toilet Support Arm (front height + 3.5in) = 31.5in
Available user height: 650mm to 780mm above floor on the low end and 740mm to 870mm in the front end. If any specific user height is required, simply position the 2 upper holes of the bracket (or the mounting template) 128 mm above the required height of the lowest part of the support arm. <u>Example:</u> Required user height (lowest part of wave) is 27.2in Installation height = 27.2in + 5in = 32.2in Front end height = 27.2in + 3.5in mm = 30.7in		

Always read and understand the mounting instruction and safety in use before mounting this product.

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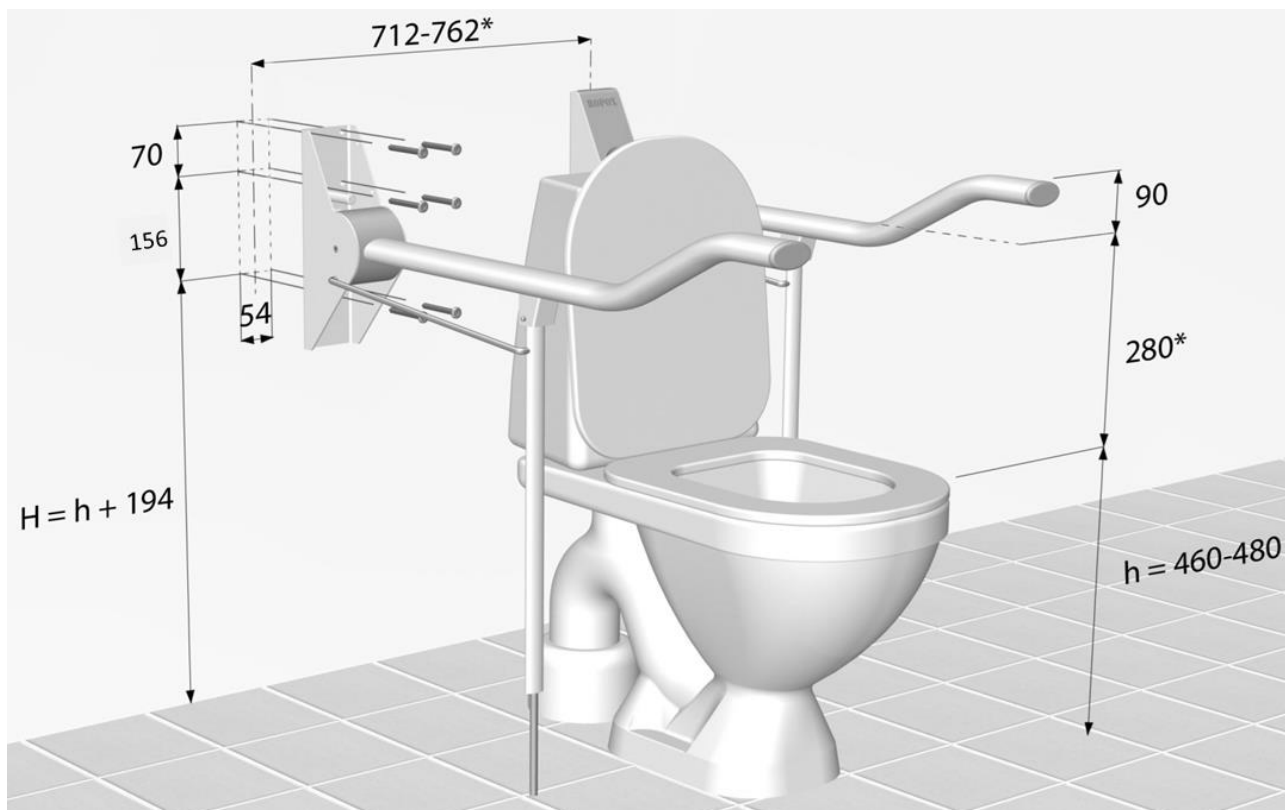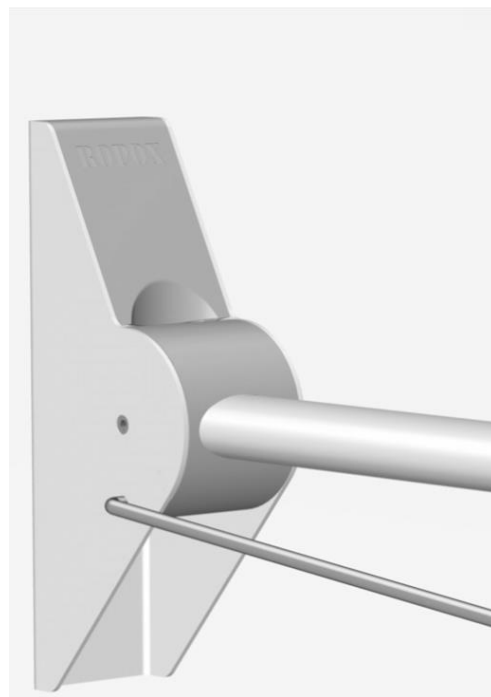
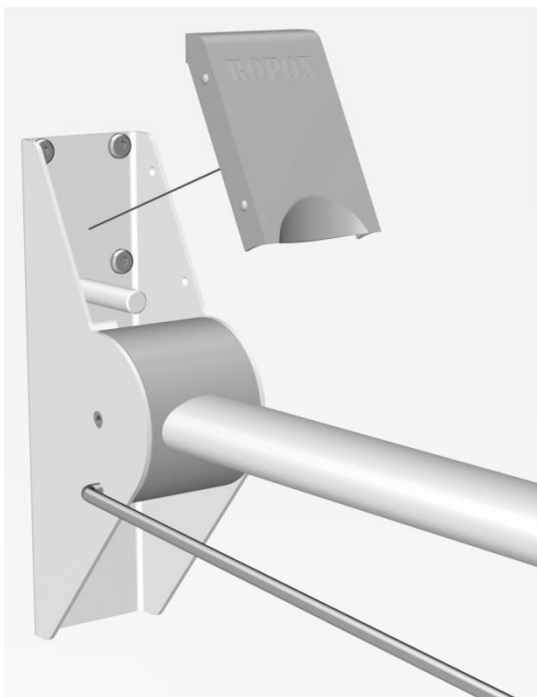
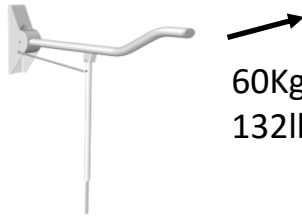
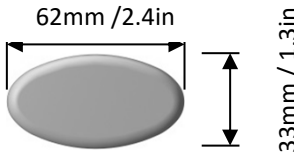
Safety in use

- Do not operate the toilet support in case of defects or damage.
- Do not overload the toilet support.
- Modifications, which may influence the operation or construction of the toilet support, are not allowed.
- Installation, service and repairs must be carried out by competent personnel.
- If the toilet support has not been assembled according to these mounting instructions the right to complain may become void.
- Only use Ropox original spare parts as replacement parts. If other spare parts are used, the right to complain may become void.

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Wall mounting

The screws and rawlplugs included are for mounting on a concrete wall. The installer must always examine the material, condition and strength of the wall and use appropriate screws and rawlplugs.

All Toilet Support Arms are tested and safe up to and including 400kg/880lbs. To comply with DS/EN 12182 the maximum user load is set to 135kg/297lbs.

Maximum pull in the top screws are 100kg/220lbs.

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Maintenance

The toilet supports are maintenance free. For safety reasons it is recommendable to check all mounting screws and moving parts to be sure all is tightened ok, and that all attachment to the wall is in correct and safe condition.

Cleaning

To clean this product, do not use a high-pressure washer. This product is not machine washable. Simply use a soft cloth or sponge with soapy water (pH 7) of max 60°C. Always clean this product after use to prevent dirt from sticking to the surface.

Important:

Do not use scouring powder or other abrasives as they may scratch the surface. Also avoid the use of cleaning utensils with sharp edges. Never use strong mineral acids or alkali.

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CE-Declaration of conformity

Ropox hereby declares that the following product

40-40125

Ropox Straight Toilet Support Arms 135

is classified as Class I and conforms to the following directive and standard:

Directives

Directive No. 93/42/EEC concerning Medical Devices (risk class 1)

Standards

DS/EN 12182: 2012	Assistive devices for persons with functional disabilities - General requirements and test methods
DS/EN 14971: 2012	Medical devices - Application of risk management to medical devices
DS/EN 9999: 2016	Assistive devices for persons with functional disabilities - Classification and terminology

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Se Generelle salgs- og leveringsbetingelser på www.ropox.dk

Complaints

See General Terms of Sale and Delivery on www.ropox.com

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ROPOX A/S